# LA FEMORO PATELLAIRE

DEPUIS 1997

PANIERS REF. 5096

# HERMES

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TRAYS #5096

## "GOALS"

- Restore a perfect stability and excellent mobility, allowing a low stress level to avoid implant loosening.
- Facilitate the patella engagement in the trochlear groove.
- Minimize wear and creep of polyethylene with optimum contact.

## A RELEVANT INNOVATIVE BIOMECHANICAL CONCEPT SINCE 1986 WITH TOTAL KNEE PROSTHESES HERMES

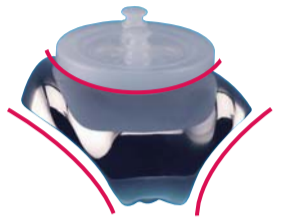
### Trochlear component:

- Anatomical trochlea in depth and alignment, facilitating patellar contact and tracking.
- Elevated outer face.
- Possible interchangeability between femoral sizes and bone preparation.


### Patellar component :

- Only one curvature radius whatever the size.
- Identical space between pegs whatever the size of the patellar component.
- Total interchangeability between patellar component size and trochlear component size.

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
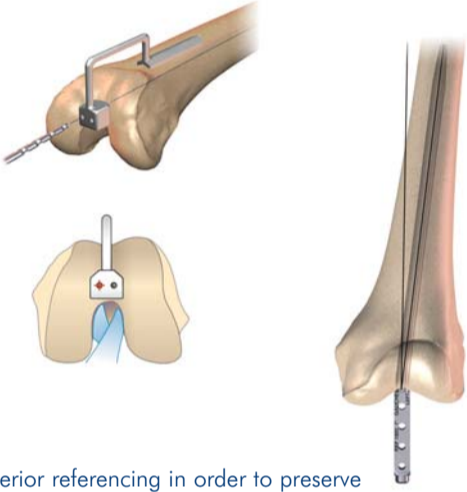
### 2 EXTERNAL ROTATION

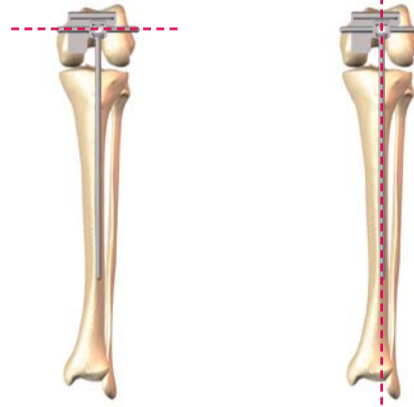
Adjusting the external rotation :  
 • Alignment with the transepicondylar axis  
 or with the tibia mechanical axis at 90° of flexion  
 • Impaction of the centromedullar rod


### 3 ANTERIOR CUT
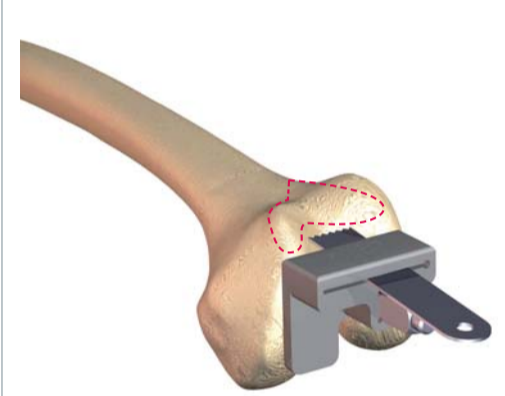
### 4 SIZE DETERMINING

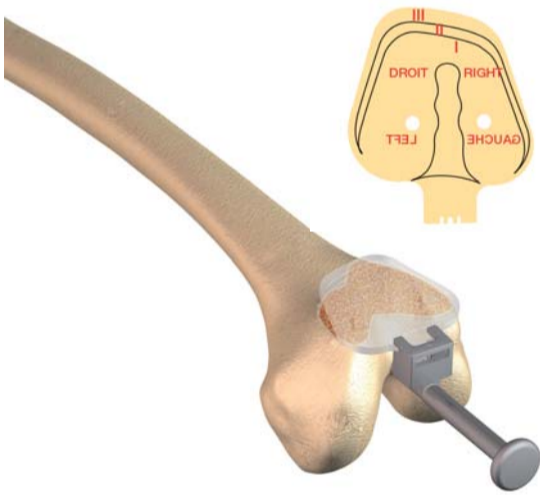
### 5 FINISHING AND CHAMFER

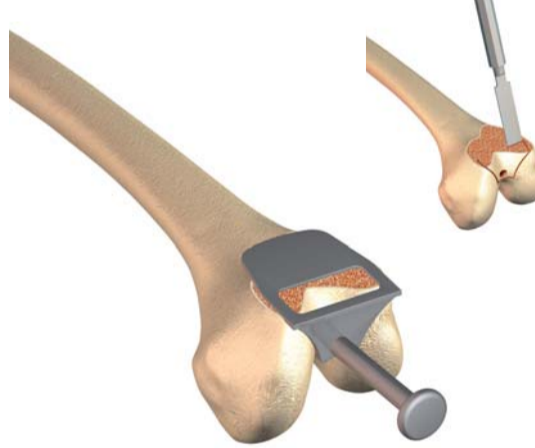One cutting bloc for all sizes

### 6 PATELLA AND TRIALS
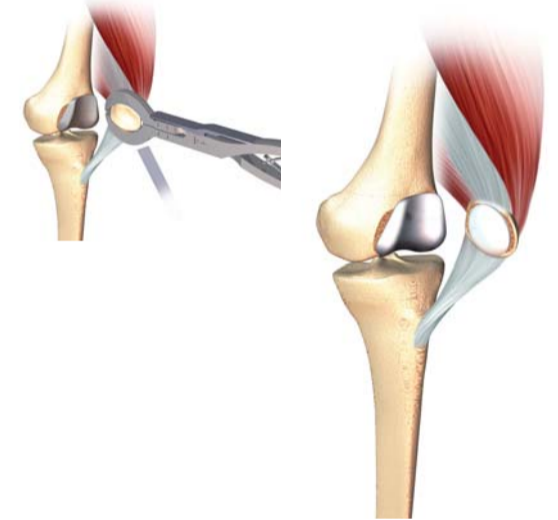
## IMPLANTS

## INSTRUMENTATION

### PATELLAR BUTTON UHMWPE\* (ISO 5834/1-2)

	Ø 31	Ø 34	Ø 36	Ø 39
Diameter (mm)	Ø 31	Ø 34	Ø 36	Ø 39
Thickness (mm)	8	9	10	11
Reference	5090	5091	5092	5093

### TROCHLEAR COMPONENT cobalt chromium alloy (ISO 5832/4)

	Size	1	2	3
Reference	Right	5901	5902	5903
	Left	5911	5912	5913

\* ALL UHMWPE implants are manufactured from compression moulded sheets and sterilised by gamma irradiation in an argon gas environment (minimum dose 25 kGy).

CERAVER - LES LABORATOIRES OSTÉAL MEDICAL  
 SIREN 2087 2897 - RUE DE LA BELLE ÉTOILE 63000 LEZ - FRANCE  
 TEL. : (+33) 1 46 66 66 66 - FAX : (+33) 1 46 66 66 99 - WWW.CERAVER.COM - CONTACT@CERAVER.COM

Copie Nord 2

TOG32/12.09