

COMPATIBILITÉ GAMME HIFIT ET GAMME HERMÈS (PS ET RÉVISION)








DIFFÉRENCES FÉMURS HiFIT / HERMÈS

- **FEMUR :**

- ❑ Pas de différences dimensionnelles (APxML) mais ajout de tailles intermédiaires.
- ❑ Compatibilité totale entre composants HiFIT et HERMÈS (PS ou Révision).

Astuce (sauf taille 9) : Dénomination taille HiFit/2 = Dénomination taille Hermès

FÉMURS HiFIT 	1	2	3	4	5	6	7	8	9
FÉMURS HERMÈS PS 		1	1,5	2	2,5	3	3,5	4	5
FÉMURS HERMÈS DE REPRISE 		1		2		3		4	






DIFFÉRENCES TIBIAS HiFIT / HERMÈS

- **TIBIA :**

- Forme de l'embase tibiale a légèrement évolué :








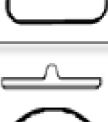
- **Conséquence : AUCUNE COMPATIBILITE** entre plateaux tibiaux HERMÈS et embases tibiales HiFIT (et inversement)

EMBASES HiFIT (AP x ML) 	A	B	C	D	E	F
	(39 x 63mm)	(42 x 66,5mm)	(45 x 70mm)	(48 x 74 mm)	(52 x 78,5mm)	(56 x 84mm)

EMBASES HERMÈS PS (AP x ML) 	0	1	2	3	4	5
	(42 x 65mm)	(42 x 70mm)	(47x 75mm)	(51,5 x 80mm)	(55,5 x 85 mm)	(60 x 90mm)
EMBASES HERMÈS DE RÉVISION (AP x ML) 		1	2	3	4	
		(42 x 70mm)	(47 x 75mm)	(51,5x80mm)	(55,5x85mm)	





COMPATIBILITÉ FÉMURS HiFIT / TIBIA HERMÈS REPRISE

		COMPOSANTS FÉMORAUX HiFIT 									ETIQUETTES
		1	2	3	4	5	6	7	8	9	
COMPOSANTS TIBIAUX HERMÈS REPRISE 	1	✓	✓	✓	✓	✓	X	X	X	X	
	2	✓	✓	✓	✓	✓	X	X	X	X	
	3 PLOT 2	✓	✓	✓	✓	✓	X	X	X	X	
	2 PLOT 3	X	X	X	X	X	✓	✓	✓	✓	
	3	X	X	X	X	X	✓	✓	✓	✓	
	4	X	X	X	X	X	✓	✓	✓	✓	



COMPATIBILITÉ FÉMURS HERMÈS REPRISE / TIBIA HiFIT

		COMPOSANTS FÉMORAUX HERMÈS DE REPRISE 			
		1	2	3	4
COMPOSANTS TIBIAUX HiFIT 	A	✓ PETIT PLOT	✓ PETIT PLOT	X	X
	B	✓ PETIT PLOT	✓ PETIT PLOT	X	X
	C	✓ PETIT PLOT	✓ PETIT PLOT	✓ GRAND PLOT	✓ GRAND PLOT
	D	✓ PETIT PLOT	✓ PETIT PLOT	✓ GRAND PLOT	✓ GRAND PLOT
	E	X	X	✓ GRAND PLOT	✓ GRAND PLOT
	F	X	X	✓ GRAND PLOT	✓ GRAND PLOT