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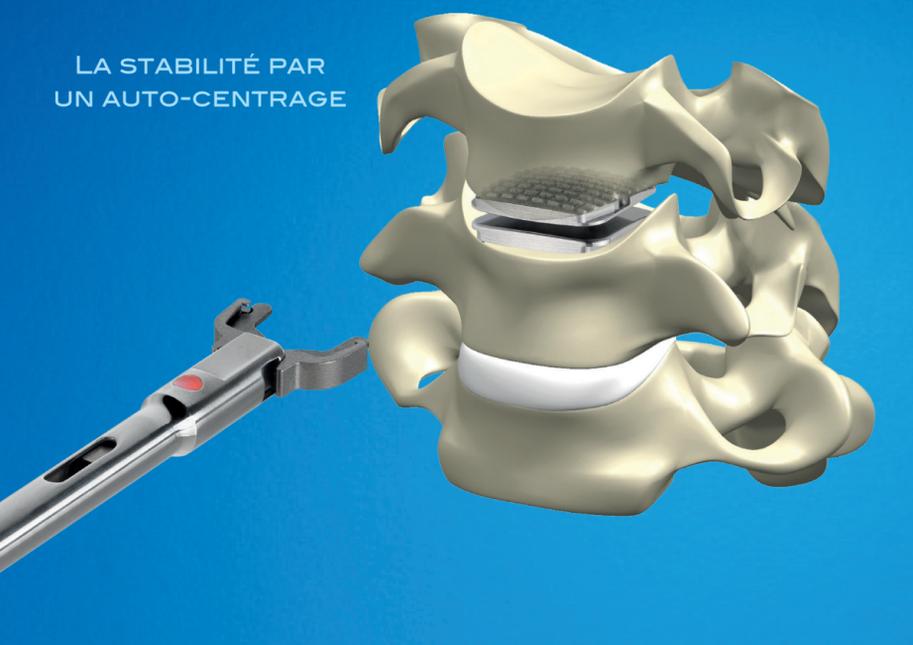
LA PROTHÈSE DISCALE CERVICALE

PANIER REF 19400

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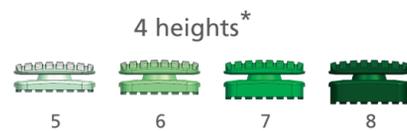
TRAY #19400

CERADISC

- Preassembled prosthesis on a clamp, connectable to the gripper handle.
- Accurate positioning and easy implant release
- Easy-to-use instrumentation

* 4 heights available for medium and large sizes only.

SURGICAL TECHNIQUE

Prior to the identification of the median plane, the surgeon has already performed the root dissection and decompression.

1 SELECTION OF ANTERO-POSTERIOR SIZE

2 MEDIO LATERAL AND HEIGHT SELECTION

medio lateral

height

3 CHOICE OF IMPLANT SIZE AND HEIGHT

Sizes	Height	5	6	7	8
SMALL 15 x 13 mm		19000	19010	19020	
MEDIUM 17 x 15 mm		19100	19110	19120	19130
LARGE 19 x 17 mm		19260	19270	19280	19290

4 CLAMP AND GRIPPER HANDLE CONNECTION*

UP

5 PRE-ASSEMBLED IMPLANT INSERTION*

6 IMPLANT RELEASE THANKS TO CLAMP REMOVAL*

* more info next page

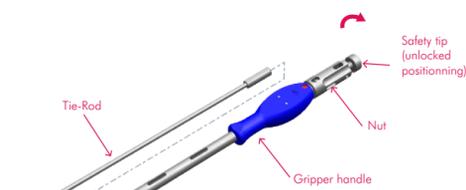
* more info next page

* more info next page

THE GRIPPER HANDLE : INSTRUCTIONS FOR USE


INSTALLATION OF THE IMPLANT ON THE GRIPPER HANDLE

- Turn the safety tip ¼ counter clockwise to set gripper handle in locked position.
- Insert the tie rod in the gripper handle, starting with the threaded portion. The guide rail allows only one position.


- Turn the nut clockwise to the line at the end of the tie rod.

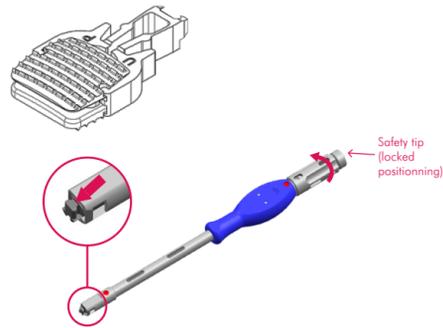- Get the previously selected implant out of its blister. It comes preassembled and held by a clamp.


In order to connect the clamp to the gripper handle :

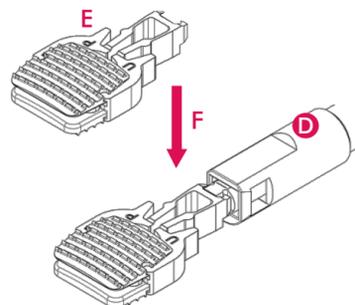- The upper face of the gripper handle is indicated by a red dot.
- Position the implant so that the word UP engraved on the clamp is visible.
- Slide the clamp over gripper handle.

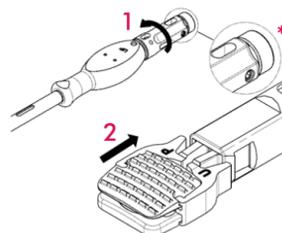
- Keep on screwing the nut until stop.
- The clamp is then firmly held by the gripper handle, and the safety remains in locked position*


The implant is now ready to be slightly impacted in the spine.

IMPLANT RELEASE

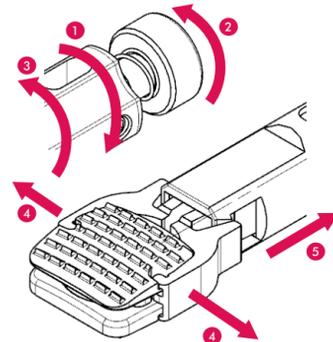- Unscrew the nut (1/8 counter clockwise) to allow the safety tip to be unlocked.
- Unlock the safety tip (1/4 turn clockwise).
- Screw the nut until its stop.
- The clamp is now opened, and the implant released.
- The gripper handle can then be softly removed.

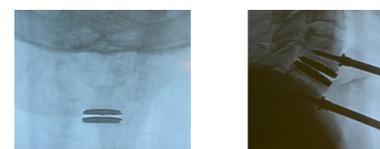- At this final stage, it may be useful to impact the upper or the lower plate using the distractor, in order to overlay the implant on the bone plates.

CERAVÉR - LES LABORATOIRES OSTÉAL MEDICAL
85, RUE DE LA BELLETOILE - 93597 ANTOISY
TEL. : (+33) 1 48 85 85 85 - FAX : (+33) 1 48 85 05 05 - WWW.CERAVÉR.COM - CONTACT@CERAVÉR.COM

Copie Nord 2

R1_CAD_309_TO_06.20